



## **SHORT-CUTS**

6	Press 2 seconds	Activation of the intensity substages 0.0 to 0.9
2	Press 5 seconds	Deactivation of the intensity substages
5 4	Press both	Display of software version
2 5	Press both	Presetting of the total number of pessure-waves
2	Press	Deactivation of the presetting of the total number
1 4	Press both	Reset pulse rate to zero (For single pulse select 0)
3 6	Press both	Reset intensity to zero
5	Press	Average value per treatment
5 3	Press both	Total pulses of the connected therapy source
5 6	Press both	Total amount of pulses emitted by the device