

Cleaning Your Equipment

- 1 Disconnect the therapy source from the control unit.
- 2 Remove the gel pad from the therapy source.
- 3 Use a damp non-scratching cloth or wipe to remove excess gel from the contact surface.
- 4 Clean the gel pad with warm water.
- 5 When necessary, use an appropriate disinfectant to clean the contact surface, gel pad, and O-ring.
- 6 Dry the contact surface, gel pad, and O-ring with a disposable cloth or clean towel.



Recommend using a damp, non-scratching cloth to remove ultrasound gel and surface debris.

Do Not Use

- ☒ Organic solvents (acetone)
- ☒ Abrasive cleaners
- ☒ Rescue™ wipes or similar wipes

Recommended Disinfectants

RECOMMENDED DISINFECTANT

Metrex CaviWipes®

CAVIWIPES ACTIVE INGREDIENTS

- Benzethonium chloride **0.28%**
- Isopropyl alcohol **17.20%**



EQUIVALENT OPTIONS

- 70% Ethyl alcohol
- 70% Isopropyl alcohol
- Light chlorhexidine solution
- Light soap

PiezoWave^{2T} VET Care

- 1 Remove gel from the therapy source between uses. Dried gel can damage the foil.
- 2 Do **not** puncture or scratch the foil.
- 3 Store all equipment in a **cool, dry location** away from sunlight.
- 4 Recommended to plug in through a **GFI power strip**. If using an extension cord, recommend **10A 125V**.

SERVICE SPECIALIST

Eddie Jentz

eddie.jentz@elvationusa.com